INSTRUCTIONS FOR USE

Intended Purpose

To transfer energy for defibrillation therapy to the heart and to provide CPR assistance/feedback.

Defibrillation Pads with CPR Device

Use Cardiac Science® defibrillation pads with the Powerheart® G5 AED for emergency treatment of sudden cardiac arrest. The AED automatically identifies electrode pads connected to it, and adjusts the maximum shock energy.

- For patients over 8 years of age or heavier than 55lbs (25kg)
- Intended for short-term use (<8hrs)
- Use before the expiration date listed on the package.
- The AED automatically reads the expiration date of the electrode pads, and notifies the user of expired pads.

WARNING! Do not reuse pads.
Reuse of pads may result in skin burns, improper AED performance, or patient-to-patient contamination.

DO NOT open until ready to use.

DO NOT use damaged pads.

Failure to remove blue plastic completely could impact therapy delivery. Before use, check cable and connectors for damage. DO NOT pull on the lead wires to separate the pads from the blue plastic.

Not intended for use in pacing.

When placing pads:
Place the pad at least an inch from any implanted device.
Ensure patient's skin is clean and dry before use.

Manufactured for:
ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA USA
01824-4105
www.zoll.com

ZOLL International Holding B.V.
Einsteinweg 8A
6662 PW Elst
Netherlands

ZOLL Medical
Switzerland AG
Baarerstrasse 8
6300 Zug, Switzerland

English

Patents pending.
For a complete list of U.S. and foreign patents,
see www.zoll.com/patents.

Not made with
natural rubber latex

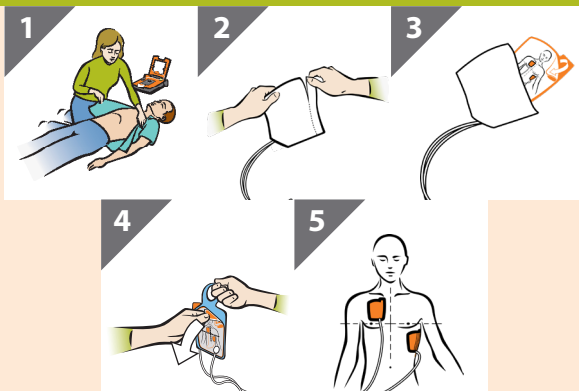
Cardiac Science, Powerheart, the Shielded Heart logo and Rescue Ready are trademarks of ZOLL Medical Corporation.

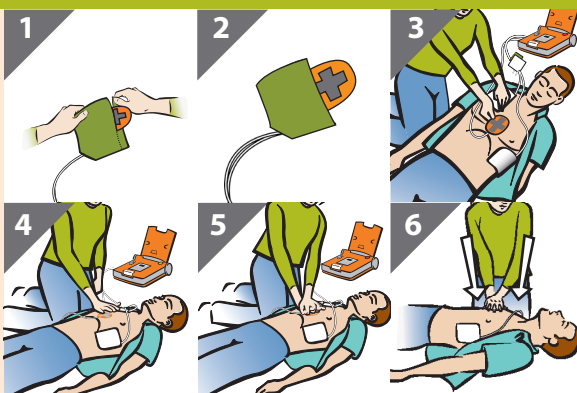
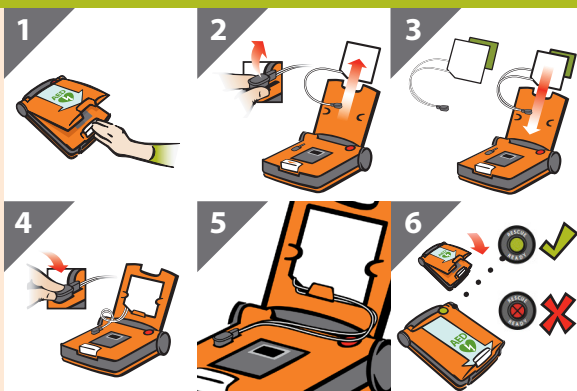
Copyright © 2024 ZOLL Medical Corporation.

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All Rights Reserved.