ES

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USO PREVISTO

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PROACT Medical Ltd

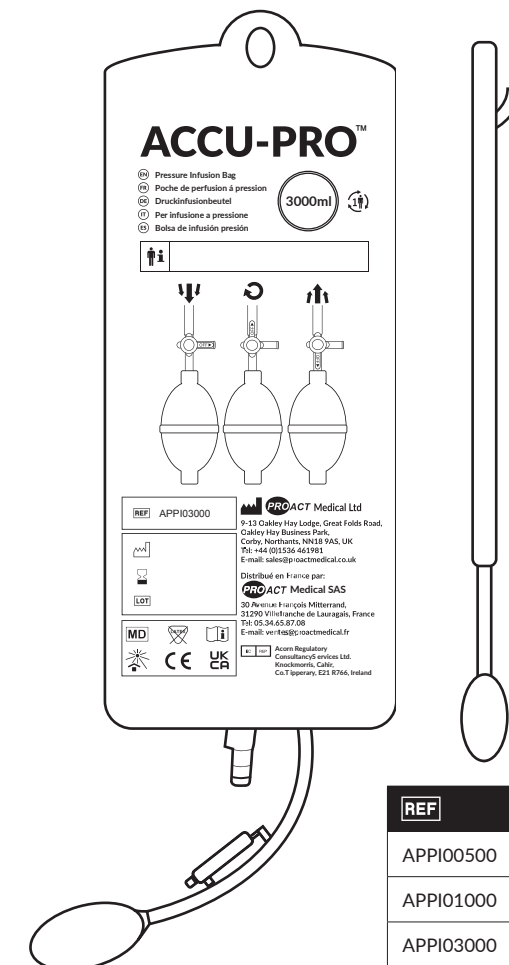
9-13 Oakley Hay Lodge, Great Folds Road,
Oakley Hay Business Park
Corby, Northants, NN18 9AS, UK
Tel: +44 (0)1536 461981
E-mail: sales@proactmedical.co.uk

EC REP

Acorn Regulatory
Consultancy Services Ltd.
Knockmorris, Cahir,
Co. Tipperary, E21 R766, Ireland

ACCU-PRO™

- EN Pressure Infusion Bag
- FR Poche de perfusion á pression
- DE Druckinfusionbeutel
- IT Per infusione a pressione
- ES Bolsa de infusión presión

IFU122-v10-2024-12

EN

INSTRUCTIONS FOR USE

INTENDED USE

- The ACCU-PRO Pressure Infusion Bags are disposable (limited reuse) bags with inflation bulb to increase the pressure of parenteral fluids to increase the rate of delivery to patients.

WARNINGS

- Use of this device requires specialist training.
- Do not expose to freezing or excessive heat conditions.
- Avoid exposure to direct sunlight or UV light.
- Bag pressure should not exceed 300mmHg.
- Colour coded pressure gauge will reflect the pressure of the fluid bag during inflation and will reduce accurately whilst fluid levels decrease
- In the event that one of these devices is involved in a serious clinical incident, please report it to your local competent authority and PROACT Medical Ltd.

PRE-USE CHECKS

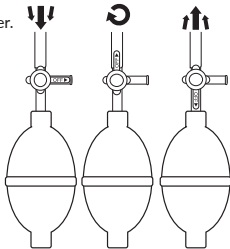
Do not use this device unless the following checks have been performed.

- Check device for completeness, discolouration, damage and flaws.
- Test inflate to full pressure before use to check for leaks.
- Ensure pressure gauge can move freely and is clear of obstructions.
- In event of pre-use check failure return product to supplier for inspection.

DIRECTIONS FOR USE

These directions are general guidelines intended for use by qualified medical personnel. Any instructions and contraindications given are not exhaustive and it is the responsibility of the clinician to ensure the safe, correct use of this product.

- Notes may be written on the infusion bag in marker pen.
- Insert fluid bag on the hook between netting and air bladder.
- Hang pressure infusion bag and fluid bag from I.V. pole.
- Turn stopcock handle to the INFLATE position as indicated in diagram.
- Pump inflation bulb until pressure gauge indicates desired pressure.
- To maintain a leak-proof seal, turn stopcock handle to the HOLD AIR position as indicated in diagram.
- Check the pressure gauge during use to ensure the infusion bag is maintaining the correct pressure
- To deflate, turn stopcock handle to the DEFLATE position as indicated in the diagram.

REPROCESSING

The ACCU-PRO Pressure Infusion Bag is sold as a disposable device that may be reused (up to a maximum of 20 times). Upon the judgement of a clinical professional with appropriate consideration for associated risks of cross contamination or infection.

Between uses the Pressure Infusion bags may be cleaned using a soft cloth with a water-soluble, mild detergent/diluted bleach solution (1-2%). Wipe the cuff with the cleaning solution then rinse and dry. NOTE: If the Pressure Infusion bag has been contaminated with blood or other fluids then it should be discarded.

DISPOSAL

The device should be disposed in accordance with local guidelines, depending on the potential risks of cross-contamination and infection at the time of disposal.

	Patient Information		Inflate		Deflate
	Single patient multiple use		Hold air		

PROACT Medical Ltd

9-13 Oakley Hay Lodge, Great Folds Road,
Oakley Hay Business Park
Corby, Northants, NN18 9AS, UK
Tel: +44 (0)1536 461981
E-mail: sales@proactmedical.co.uk

EC REP

Acorn Regulatory
Consultancy Services Ltd.
Knockmorris, Cahir,
Co. Tipperary, E21 R766, Ireland

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PROACT Medical SAS