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**Dátum poslednej revízie: 02/2018**

medical devices, sterilization by irradiation

www.bioster.cz, www.traumacel.cz

### **Instruction information. Read carefully! (EN)**

Manufacturer: BIOSTER, a.s.,  
Tejny 621, 664 71 Veverská Bítýška  
Czech Republic

### **Rescue - Burns**

(COLOR ORANGE-RED)

#### **Characteristics**

RESCUE – BURNS kit is set consisting of specialized devices for burns treatment in the course of first aid. Is universal, it can be used by doctor and laic for the temporary treatment of large superficial injury, mainly skin, hypoderm and burns of all degrees.

All components are individually packed and content the information about the manufacturer, product, badge number, date of use, information whether the product is sterile and information for use.

#### **Packaging**

non-sterile gloves, sterile gloves, isothermic foil to prevent thermal loss, saline solution to wash injured areas, bandaging material to fix dressing materials, highly absorbent dressing, and a sterile emergency burn dressing to cover extent injured area.

#### **Application**

Injury treatment procedure:

- wear non - sterile gloves
- remove only parts of the clothes and subjects, which are easy to remove
- wash the affected area with saline solution
- wear sterile gloves
- cover affected areas with sterile dressing, in case of a larger injury cover less affected areas with sterile emergency burn dressing
- fix the dressing with Omnisilk plaster to the healthy skin (body and head) or with bandage (limbs and head)
- use isothermic foil to prevent thermal loss or overheating when the injured person is immobilized

#### **Warnings**

The product is intended for single use only. Do not use the product if the packaging is damaged.

#### **Storage and shelf life**

The product does not require any special storage conditions. Do not use RESCUE – BURNS kit after the shelf-life date, which is stated on the package. The shelf-life date is set in accordance with a component with the earliest shelf life date.

#### **Disposal**

All retained biological material will be disposed of as the biological waste.

**Date of last revision: 02/2018**

medical devices, sterilization by irradiation

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- Výrobce: B Braun, Melsungen AG, Melsungen 342 09, Germany