INSTRUCTIONS FOR USE

Pediatric Defibrillation Pads

Use with Cardiac Science® Powerheart® G5 AED for emergency treatment of sudden cardiac arrest.

The AED automatically identifies electrode pads connected to it and adjusts the maximum shock energy.

- For children up to 8 years of age or up to 55lbs (25kg)
- If child appears older or heavier, use non-pediatric defibrillation pads. Do not delay treatment to determine child's exact age/weight.
- Intended for short-term use (<8 hrs)
- Use before expiration date listed on package.
- The AED automatically reads the expiration date of the electrode pads, and notifies the user of expired pads.

WARNING! Do not reuse pads.

Reuse of pads may result in skin burns, improper AED performance or patient-to-patient contamination.

DO NOT open until ready to use.

DO NOT use damaged pads.

Failure to remove blue plastic completely could impact therapy delivery. Before use, check cable and connectors for damage.

DO NOT pull on the lead wires to separate the pads from the blue plastic.

Not intended for use in pacing.

WARNING! Not for adult use.

Pediatric defibrillation pads are for use only with children 8 years of age or younger, or 55lbs (25kg) or lighter.

When placing pads:

Place the pad at least an inch from any implanted device.

Ensure patient's skin is clean and dry before use.

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English

Patents pending.

For a complete list of U.S. and foreign patents, see www.cardiacscience.com.

Not made with
natural rubber latex

2797


Cardiac Science, Powerheart, the Shielded Heart logo and Rescue Ready are trademarks of Cardiac Science Corporation.

Copyright © 2021 Cardiac Science Corporation.


All Rights Reserved.

INSTALL PEDIATRIC PADS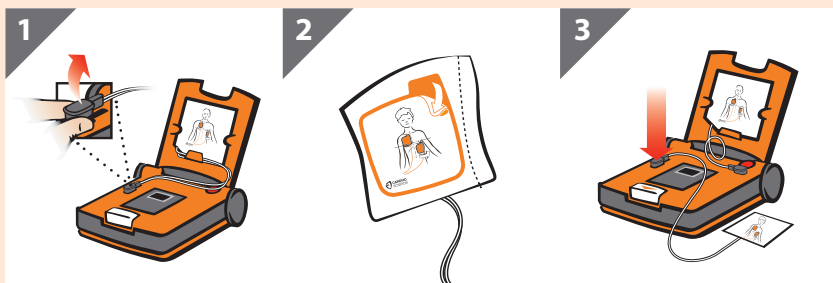


HOW TO APPLY PADS

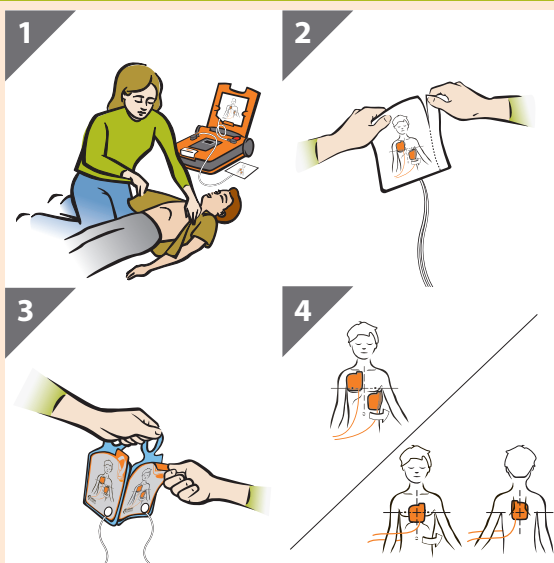AFTER THE RESCUE

RECONNECTING THE NON-PEDIATRIC PADS

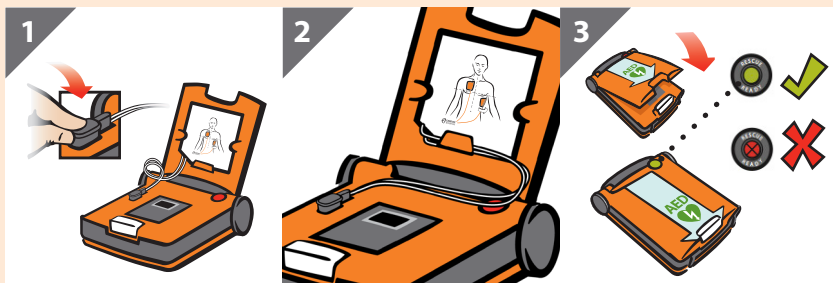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MDSS GmbH