Standard Violet and Blue Skin Markers

STERILE R

Rx ONLY

MD

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

2797

English

Intended Use

Skin marker is used for surgical marking on intact skin

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions

- The skin marker ink must be applied to intact, dry skin before applying prep solution. The skin marker ink will not adhere to prep solution or wet skin.
- Prepping the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- This device is single use only. Reuse of device could result in infection/contamination and/ or device failure which could lead to patient harm.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.
- Not for ocular use.
- Device is intended to write for up to 24 feet.
- Do not use product if product is damaged or particulate is found in sterile packaging

Instructions for Use

- Remove the pen cap to expose the pen tip.
- Mark desired areas on skin with the skin marker according to facility protocol.
- Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
- Apply prep solution as instructed by prep solution manufacturer.
- Dispose or recycle general surgical devices per facility protocol.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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