




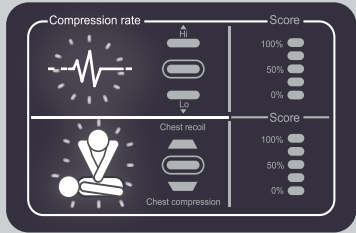
## FAQ

### MONITOR

If you cannot pair the equipment following the steps provided, you should reset the monitor.

Monitor reset: Press the "Adult-Child" button for 7 seconds approx. until the rate  and compression  icons flash once.

Turn off the monitor  and begin the pairing process again.



### BATTERY

The batteries supplied with the Practi-Man Plus equipment should be replaced when the red light appears on the monitor indicator.

The use of uncharged batteries with either the monitor or the torso impedes the proper functioning of the Practi-Man Plus.

There are no interference implications using rechargeable batteries.








# PRACTI-MAN PLUS

## ELECTRONIC CPR TRAINING

### MONITOR MANUAL



-  WIRELESS MONITOR
-  ELECTRONIC FEEDBACK
-  2 IN 1 - ADULT & CHILD FUNCTION
-  BLIND MODE
-  TEST REPORT AFTER TRAINING

### CLEANING

Do not use liquids to clean inside the manikin. Do not submerge any parts.

### BATTERY

Use consumer grade carbon zinc or alkaline 9V batteries. Batteries must be replaced with correct type. Remove batteries if product is not to be used for an extended period.

### SYMBOL GLOSSARY



PRACTI-MAN PLUS. This product is in compliance with the essential requirements of Council Directive 2014/53/EU on radio equipment (RED), and Council Directive 2011/65/EU on Restriction on the use of certain hazardous substance (RoHS).



Dispose of in accordance with your country's recommendations.

### WASTE HANDLING

PRACTI-MAN PLUS contains electronic components. Dispose of it at the applicable recycling facility in accordance with local regulations.



European directive 2012/19/EU (WEEE) WEEE: this appliance is marked according to the European directive 2012/19/EU on Waste Electrical and Electronic Equipment (WEEE). By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

The symbol on the product, or on the documents accompanying the product, indicates that this appliance may not be treated as household waste. Instead it shall be handed over to the applicable collection point for the recycling of electrical and electronic equipment. Disposal must be carried out in accordance with local environmental regulations for waste disposal. For more detailed information about treatment, recovery and recycling of this product, please contact your local city office, your household waste disposal service or the PRACTI-MAN PLUS representative where you purchased the product.

MEETS  
AHA/ERC  
FEEDBACK  
REQUIREMENTS

# PRACTI-MAN PLUS

ELECTRONIC CPR TRAINING

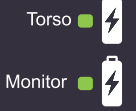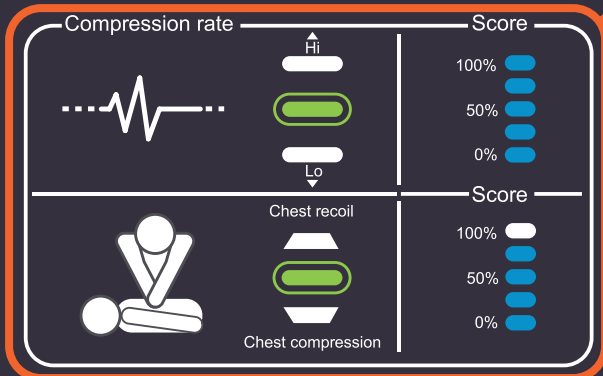



## KEY FUNCTIONS

### COMPRESSION FEEDBACK

#### RATE





#### DEPTH AND RECOIL

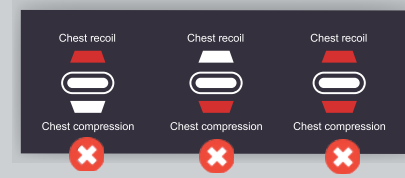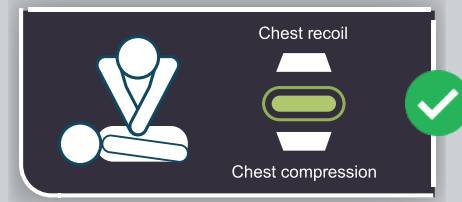

### TEST REPORT




WAIT  
10 SECONDS

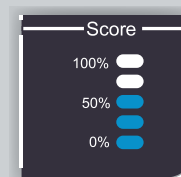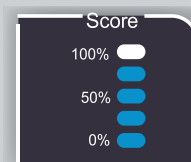

## SETTING UP

Insert batteries into the torso and the monitor (included in each Practi-Man Plus pack).

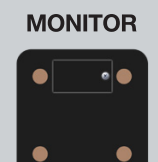

## PAIRING



Switch on the monitor and press the PRACTI-MAN PLUS torso until you hear the double beep sound and the "TORSO" light turns green. PRACTI-MAN PLUS is now ready to begin the practice.


When switching on the monitor, the default always activates on adult mode.

## ADDITIONAL FUNCTIONS

### ADULT-CHILD FUNCTION

Allows you to select standard or blind mode for both adult and child distinguished by a steady or flashing light.

#### STANDARD MODE

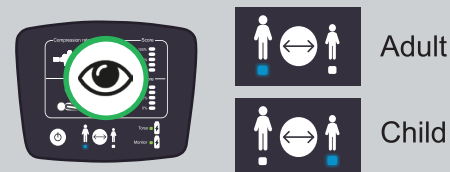

#### BLIND MODE



Tap the button until you get a steady light in your preferred option (Adult - Child).

Tap the button until you get the flashing light in your preferred option (Adult - Child).

In blind mode, the feedback is not visible during practice. However ten seconds after the session has finished you can view the test report as well as in standard mode.